



Proven Effective

Scar Fx[®] Silicone Sheeting



RejûvaSil® Silicone Scar Gel









Scar Esthetique® Scar Crème







30 Years60 Countries4 Million Scars

Why choose the Scar Heal® product line?

According to the International Advisory Panel on Scar Management, silicone is the gold standard in non-invasive scar treatment. Silicone occlusion enhances the skin's own natural healing process to reduce collagen production and help minimize the appearance of problem scarring.

Our Scar Heal® product line is made up of three silicone-based products proven to help manage the appearance of even the most difficult scars. For over 30 years, these products have been recommended by physicians in over 60 countries.

RejûvaSil® Silicone Scar Gel

- 97% Liquid Silicone
- Softens and Flattens Raised Scars
- Reduces Redness and Discoloration
- Relieves Itchiness and Discomfort
- Suitable for Children and Facial Scars





Scar Fx® Silicone Sheeting

Softens and Flattens Raised Scars

Rejûvaskin'

Scar Fx®

- 100% Healthcare Grade Silicone
- Soft, Pliable, and Contours Easily to the Body
- Self-adhering Sheets in 13 Sizes (4 Specific to the Breast)
- Can be Washed and Reused for 30-45 days

Scar Esthetique® Scar Crème

- 23 Effective Ingredients including Silicone
- Targets Discoloration on the Cellular Level
- Easily Absorbed and Non-greasy
- Recommended for Long-term Scar Management
- Polypeptides, Anti-oxidants, and All-natural Botanicals

66

We have strongly recommended Scar Heal products for our plastic surgery patients for the past 5 years. With optimal surgical techniques along with Scar Heal products we are able to obtain the best results possible. They are user friendly and are an affordable, reliable and safe treatment."

- Dr. David Halpern, MD

Physician Recommended Scar Management System



Scar Fx at Night

At night place Scar Fx over your clean and dry scar for 8-12 hours. The occlusive barrier provided by the silicone sheet helps create the ideal healing environment for your scar.



RejûvaSil During the Day

During the day apply RejûvaSil 2-3 times to help maintain the healing environment. RejûvaSil also helps relieve itchiness and discomfort associated with scars.



Scar Esthetique for Discoloration

Once your scar is flat and smooth, apply Scar Esthetique 3-4 times daily to target remaining discoloration. Many patients see continued results through longterm use of Scar Esthetique.

Products also Available in 25 Surgery-specific Scar Heal® Kits

Rejûvaskin®



Scar Fx Sizes

Our Scar Fx Silicone Sheeting comes in 13 unique sizes to best fit your scar management needs. There are 9 rectangular sheets and 4 sheets specific to the breast. Our silicone sheets can be cut to any size by the end user. All sheeting is sold individually, unless it is purchased as part of a Scar Heal® Kit. Breast sizes are sold as pairs in each kit.

RECTANGULAR SIZES	CODE	POTENTIAL APPLICATION	
1.5x3" (3.75x7.5cm)	11503	Small scars, Mohs surgery, trocar sites	
1.5x5" (3.75x12.5cm)	11505	Small scars, Mohs surgery, trocar sites	
1.5x9" (3.75x22.5cm)	11509	C-Sections, Abdominal surgery	
1x12" (2.5x30cm)	10112	C-Sections, Abdominal surgery	
1x22" (2.5x55cm)	10122	Abdominoplasty, Bariatric surgery	
3x5" (7.5x12.5cm)	10305	Burns, Traumatic Scars, Breast Procedure	
4x8" (10x20cm)	10408	Burns, Traumatic Scars	
10x12" (25.4x30.5cm)	11012	Burns, Traumatic Scars	
6x48" (15x122cm)	10648	Velcro Belt with Sllicone	

BREAST SIZES	CODE	POTENTIAL APPLICATION	IMAGE
3" Breast Circle	10000	Mastopexy	
Breast Piece	11234	Augmentation	
Lollipop Breast Piece	13456	Mastopexy or Reconstruction	
Breast Anchor	15678	Augmentation or Reduction	